

## Dräger DrugTest® 5000 Test Kit

The Dräger DrugTest® 5000 system consists of two main components: the DrugTest 5000 Test Kits and the DrugTest 5000 Analyzer. It is a fast and highly accurate system for testing saliva samples for substances such as amphetamines, designer amphetamines, opiates, cocaine and its metabolites, benzodiazepines, cannabinoids, ketamine or methadone.



The DrugTest 5000 Test Kits are ergonomically designed, easy to handle and immediately ready for use. They consist of a test cassette and a sample collector. The test kit can be handled with or without gloves by right-handed or left-handed persons. The test result is clearly displayed, making misinterpretation impossible. Made from a rigid porous material with capillary properties, this non-invasive test allows hygienic handling. Any contact with the sample and its components is prevented.

### The test is performed in five simple steps, excluding incorrect operation.

The test itself is very straightforward: The operator removes the protective cap from the sample collector and hands the test cassette over to the subject, who puts the collector into the mouth and moves it back and forth between the cheek and the gum to extract a saliva sample.

When the collector has absorbed enough liquid, the integrated indicator will turn blue and the test cassette can be returned to the operator. The operator, who supervises the entire sampling process, then inserts the test cassette and the cartridge into the Analyzer. After a few minutes, the result will be clearly displayed. In order to facilitate matching of samples to subjects, the test cassette can be labelled with the subject's name and the date.

If required, the Dräger DrugTest 5000 can be used in conjunction with the saliva sample collector Dräger DCD 5000. This allows taking a second sample as well as storing and transporting the samples for laboratory analysis. If used in combination with the Dräger SSK 5000, it takes only a few minutes to determine whether a suspicious surface, preparation or substance may be associated with one of the designated types of narcotics.



D-31774-2011

**The Dräger DrugTest 5000 Test Kit:**  
easy to use, fast, reliable and secure in testing for drugs

## ORDERING INFORMATION

## Designation

Designation	Order no.
Each Dräger DrugTest® 5000 test kit consists of a sample collector test cassette and a cartridge, packaged in a foil pouch.	
Dräger DrugTest 5000 Training Test Kit	83 23 640
Dräger DrugTest 5000 Test Kit COC-OPI-BZO-THC-AMP-MET*	83 19 830
Dräger DrugTest 5000 Test Kit COC-OPI-THC-AMP-MET*	83 23 439
Dräger DrugTest 5000 Test Kit COC-OPI*	83 19 990
Dräger DrugTest 5000 Test Kit COC-OPI-BZO-THC-AMP-MET-MTD*	83 23 156
Dräger DrugTest 5000 Test Kit COC-OPI-BZO-THC-AMP-MET-MTD-KET*	83 23 646**
Packaging: 20 complete test kits per package	

\*COC: cocaine – \*OPI: opiates – \*BZO: benzodiazepines – \*THC: tetrahydrocannabinol (cannabis) – \*AMP: amphetamine – \*MET: methamphetamine + designer amphetamines –


\*MTD: methadone – \*KET: ketamine

\*\*Only sold within the European Union


## Accessories

Dräger DCD 5000 Saliva Sample Collector	83 19 910
Dräger SSK 5000 Surface Test Kit	83 20 490


## DRÄGER DRUGTEST® 5000 – THE TEST IS PERFORMED IN 5 SIMPLE STEPS:

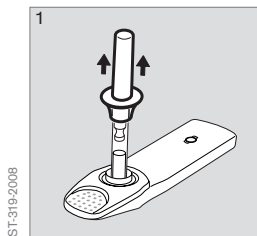


1 The protective cap is removed

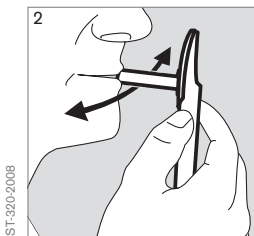


2 The sample collector is inserted into the mouth

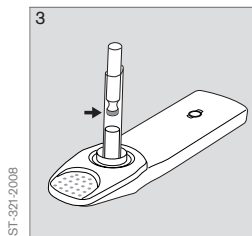

3 The integrated indicator turns blue

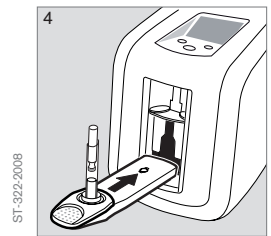

4 Test cassette and cartridge are inserted into the Analyzer

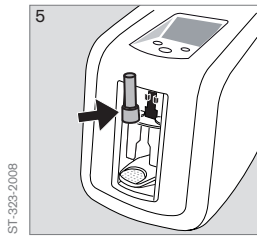

5 The display shows a clear result

## Storage and shelf life

Test kits must be stored at temperatures between 4 and 30 °C (39 to 86 °F) in their original foil pouch packaging. Do not use a test kit if the enveloping foil pouch is broken and has, for example, holes or tears. Immediately use the test kit after opening the packaging. The test kit must not be used after the expiration date, which is clearly printed on the foil pouch.

## CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany

www.draeger.com

## Manufacturer:

Dräger Safety AG & Co. KGaA  
Revalstraße 1  
23560 Lübeck, Germany

## AUSTRALIA

Dräger Safety Pacific Pty. Ltd.  
8 Acacia Place  
Notting Hill VIC 3168  
Toll-free +61 1800 DRAGER  
(1800 372 437)  
Fax +61 1800 64 74 84  
customer.service@draeger.com

## CANADA

Draeger Safety Canada Ltd.  
2425 Skymark Avenue, Unit 1  
Mississauga, Ontario, L4W 4Y6  
Tel +1 905 212 6600  
Toll-free +1 877 Dräger 1  
(+1 877 372 4371)  
Fax +1 877 651 0902  
Toll-free Fax +1 800 329 8823

## P. R. CHINA

Draeger Safety Equipment  
(China) Co., Ltd.  
A22 Yu An Rd, B Area,  
Tianzhu Airport Industrial Zone,  
Shunyi District, Beijing 101300  
Tel +86 10 80 49 80 00  
Fax +86 10 80 49 80 05

## FRANCE

Dräger Safety France SAS  
3c route de la Fédération, BP 80141  
67025 Strasbourg Cedex 1  
Tel +33 3 88 40 76 76  
Fax +33 3 88 40 76 67

## MEXICO

Draeger Safety S.A. de C.V.  
Av. Peñuelas No. 5 Bodega No. 37  
Fraccionamiento Industrial  
San Pedro  
Querétaro, Qro México  
Tel +52 442 246-1113  
Fax +52 442 246-1114

## NETHERLANDS

Dräger Safety Nederland B.V.  
Edisonstraat 53  
2700 AH Zoetermeer  
Tel +31 79 344 46 66  
Fax +31 79 344 47 90

## REGION ASIA PACIFIC

Dräger Safety Asia Pte Ltd.  
67 Ayer Rajah Crescent #06-03  
Singapore 139950  
Tel +65 68 72 92 88  
Fax +65 65 12 19 08

REGION CENTRAL  
AND SOUTH AMERICA

Dräger Panama Comercial S. de R.L.  
Complejo Business Park,  
V tower, 10th floor  
Panama City  
Tel +507 377-9100  
Fax +507 377-9130  
contactcsa@draeger.com

## REGION MIDDLE EAST, AFRICA

Dräger Safety AG & Co. KGaA  
Branch Office  
P.O. Box 505108  
Dubai, United Arab Emirates  
Tel +971 4 4294 600  
Fax +971 4 4294 699  
contactuae@draeger.com

## SPAIN

Dräger Safety Hispania S.A.  
Calle Xaudaró 5  
28034 Madrid  
Tel +34 91 728 34 00  
Fax +34 91 729 48 99

## UNITED KINGDOM

Dräger Safety UK Ltd.  
Blyth Riverside  
Business Park  
Blyth, Northumberland  
NE24 4RG  
Tel +44 1670 352-891  
Fax +44 1670 356-266

## USA

Dräger Safety, Inc.  
101 Technology Drive  
Pittsburgh, PA 15275  
Tel +1 412 787 8383  
Fax +1 412 787 2207